Matriz Cervical

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Cervical Matrix

Transparent matrix for Class V restorations.

Matriz Cervical is an instrument used for modelling composites in Class V restorations, composed by five different sizes and flexible shell shapes. The matrices provide a perfect subgingival adaptation in the cervical region of the tooth when kept under pressure, besides allowing the visualization and polymerization of the composite. Transparent, the matrix is an excellent tool anatomy and reduced time in the procedure.

Indications: Cervical Matrix is indicated for modelling composites in Class V restorations, assuring a perfect anatomy in the cervical region.

Shapes: Available in 5 sizes.

Use: Selection among the five available sizes the one which better adjusts to the tooth to be restored. Adjust the matrix to the instrument. If the cavity has a subgingival extension, promote the gingival retraction. Insert the last layer of composite, compress the matrix against the tooth forcing the composite to flow to incisal/occlusal region. Remove the excesses from the edges of the matrix and polymerize for 10 seconds through the matrix. Remove the matrix and complete the polymerization according to the recommendations of the manufacturer of the composite. Perform the finishing and polishing procedures.

Warnings, precautions and restrictions: Professional use. The matrices are for single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. Any serious incident related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Cleaning and sterilization: The instruments are autoclavable (126°C - 16min / 134°C - 18min). Before sterilization, proceed with the cleaning. Avoid the use of enzymatic detergent. Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure water between 6,5 - 7,5. Prepare the solution according to the manufacturer's instructions. The water should be potable. For a better effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the products and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the pieces with absorbent paper or disposable wipes. Inspection: Check the product before each use, observing whether there is damage to the tips or cracks along the instrument, and do not use in such cases. Cracks are indicative of material fragility, which can cause the product to break.

Composition: Instruments: Polysulfone. Matrizes: Polypropylene.

Ref.	Code	Content
04210	000504000	2 instruments + 10 matrices assorted
04220	000504039	Refill assorted - 100 units
04225XS	000504023	Refill extra small - 50 units
04225S	000504024	Refill small - 50 units
04225M	000504025	Refill medium - 50 units
04225L	000504026	Refill large - 50 units
04225XL	000504027	Refill extra large - 50 units

Ref.	Code	Contenu
04210	000504000	2 instruments + 10 matrices assorted
04212	000504016	Recharge 100 matrices assorties

Manufacturer:
TDV Dental Ltd.
CNPJ: 18.591.788/0001-60
V. de Novembro, 9944 | CEP 89107-000
Pomerode/SC - Brasil
Resp. Téc. Tais Cugato Coelho Pereira
CFFSC 13884
Reg. ANVISA 10291220051
tel: 0800 47 1020 | fax: 47 3395-6115
info@tdv.com.br www.tdv.com.br

EC REP
O.B.V.S. S.R.L.
Central Walling 53
1000 Brussels, Belgium
Tel: +32 27 375 00 00
E-mail: mail@obvts.net

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Composition: Instruments: Polysulfone. Matrizes: Polypropylene.

Content
2 instruments + 10 matrices assorted
Refill assorted - 100 units
Refill extra small - 50 units
Refill small - 50 units
Refill medium - 50 units
Refill large - 50 units
Refill extra large - 50 units

Contenu
2 instruments + 10 matrices assorties
Recharge 100 matrices assorties

Version 03 - 03/2020

Estérilizable/Esterilizável/Sterilizable/Estérilizable (Autoclave 126°C - 16min / 134°C - 18min)

Matrices Instruments / Instruments

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