Structur® 3

Carefully read instructions prior to use

Product description:

Structur 3 is a cold-polymerizing paste-paste system based on multifunctional methacrylic acid esters for the fabrication of temporary restorations.

Shades:

Structur 3 in the cartridge: A1, A2, A3, A3.5, B1, B3, C2, BL
Structur 3 in the QuickMix syringe: A1, A2, A3, B1

Indications:

- Fabrication of temporary crowns, bridges, inlays, onlays, partial crowns, veneers and temporary posts
- Fabrication of long-term temporary restorations
- Rebasement of prefabricated temporary crowns made of composite, polycarbonate or metal

Contraindications:

Structur 3 contains methacrylates, amines, terpenes, benzoyl peroxide, BHT and should not be applied in cases of known allergies to these ingredients.

Type of application:

Preparation:

Structur 3 in the cartridge:

Insert **Structur 3** cartridge into the **VOCO** dispenser (type 2) or a comparable dispenser. For technical reasons, there may be slight differences in the filling levels of the two cartridge chambers prior to initial use. In this case, remove the cartridge cap and carefully squeeze the material out until it is delivered evenly from both openings. Then attach a type 6 mixing tip and lock with a 90° turn clockwise. The material is extruded by pressing the lever and is automatically mixed in the correct ratio. After use, do not remove the mixing tip from the cartridge. Do not replace the used mixing tip with a new one until immediately before the next application. When changing the mixing tips, check the openings of the cartridge to ensure that the material can flow freely.

Structur 3 in the QuickMix syringe:

Remove the cap from the syringe by turning it anti-clockwise. Attach a type 10 mixing tip. The slots on the syringe and the mixing tip must correspond. The mixing tip is fixed by rotating it clockwise through 90°. The material is automatically mixed in the correct ratio in the tip as it is dispensed and can be applied immediately. All QM syringes come with a detachable piston. The piston can easily be pulled backwards out of the syringe and then reinserted. The seals in the syringe do not change their position while this is being done. This prevents cross-contamination of the material in the syringe.

Processing directions:

First level out any undercuts in the preparation using a non-eugenol cement. Take an impression if not using prefabricated dental copings or similar restorative materials. Prepare the impression by leveling undercuts and cutting drainage channels. If the wall thickness of the temporary restoration is expected to be too thin, enlarge the impression, paying particular attention to removing the septums in the interdental embrasures. In case of gaps in the molar region, cut a groove in the impression to maintain a stable, bar-shaped connection in the temporary restoration. Then clean and air dry the impression.


Apply **Structur 3** directly into the impression, applying the material initially to the occlusal surfaces of the impression and filling up towards the gingival areas. To prevent the formation of bubbles during application, always keep the opening of the mixing tip immersed in the material. Insert the impression into the mouth no later than 30 seconds after commencement of mixing.

Remove the temporary restoration from the mouth during the elastic phase i.e. within 1 to 1.5 minutes after beginning to mix.

The setting process should additionally be monitored by observing the excess material in the **mouth**.

The setting time of material (in mixing tips) that has not been used cannot be compared to the setting time of the material in the mouth.

The working times at a room temperature of 23 °C and 50% humidity are as follows:

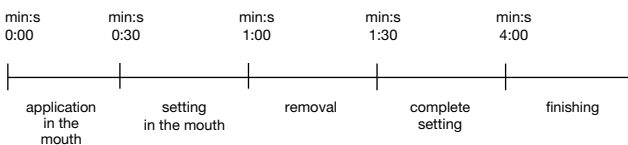Different room temperatures may result in variations from the above times. Higher temperatures shorten and lower temperatures extend these times.

Finishing:

Extract the temporary restoration from the impression after 4 minutes at the earliest and completely remove the inhibition layer, caused by oxygen in the air, by wiping with alcohol (e.g. ethanol).

Then carefully grind any excess material and rough areas near the gingiva. If necessary, finish the temporary restoration with fine, cross-cut carbide burs. If required, polish the temporary restoration manually (e.g. with a rubber polisher, fine sandpaper or goat hair brush).

Luting:

Carefully clean and dry the core, if necessary removing any insulating material. Attach the finished temporary restorations with a temporary luting cement. Eugenol cements can inhibit the setting of luting composites that may be used later at the permanent luting stage. If a luting composite is to be used for permanent luting, attach the temporary restoration with a eugenol-free cement.

Repairs, rebasing and customizing:

The following procedures are recommended for customization, repairs or rebasing:

Repair of defective temporary restorations:

- **Air bubbles:** fill with **Structur 3**.
- **In the case of a fracture of a newly made temporary restoration:** re-connect the fracture with freshly mixed **Structur 3**.

- **Repair of temporary restorations that have already been worn:**

Roughen the fracture with a bur and place undercuts. Apply bond (e.g. **Solobond Plus Adhesive**) to the prepared fractures and cure according to the respective instructions for use.

Apply a flowable material to the prepared areas (**GrandioSO Flow** is suitable for this purpose because it provides optimum shade matching). Reassemble the fractured pieces and cure according to the instructions for use.

Customization:

Apply a flowable material (e.g. **GrandioSO Flow**) in the required shade to the freshly prepared temporary restoration and then cure and finish according to the instructions for use.

Rebasing prefabricated temporary crowns:

Apply **Structur 3** to the prefabricated, temporary crown, following the respective manufacturer's instructions for use.

- Place the prefabricated crown on the prepared core.
- Remove the prefabricated crown from the mouth during the elastic phase, i.e. 1 to 1.5 minutes after commencement of mixing.
- In addition, monitor the setting process by observing the excess material in the mouth. The setting time of material (in mixing tips) that has not been used **cannot** be compared to the setting time in the mouth.
- Remove excess material after 4 minutes at the earliest and finish.

Indications/precautions:

- Do not inhale the dust created during the finishing of the temporary restoration. Wear a mask and protective goggles and use an extraction system.
- Before the temporary restorations are made, the prepared cores can be insulated to prevent hypersensitivity (by using fluoride varnishes, core varnishes and the like).
- Ensure that methacrylate-based linings and core build-ups (e.g. composites, compomers, Ormocer®, resin-reinforced glass ionomer cement) are adequately insulated (e.g. with glycerine) to prevent the temporary restoration becoming bonded to the core build-up.
- Inadequate hygiene or the intensive use of certain mouth rinses may lead to shade variations.
- Do not start to finish the temporary restoration until the material has completely set (approximately 4 minutes after commencement of mixing).
- If necessary, (in the case of a long-term temporary restoration) the temporary restoration should be checked and, where required, replaced.
- The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Storage:

Store at temperatures of 4 °C to 23 °C. Store cartridges protected from light i.e. in closed packages, drawers etc.

Do not use after expiration date.

Product Ordering Information:

Intro kit 50ml cartridge A2, mixing tips type 6, dispenser type 2	REF 2500
Intro kit 50ml cartridge A3, mixing tips type 6, dispenser type 2	REF 2501
pack, shade A1 50ml cartridge, mixing tips type 6	REF 2503
pack, shade A2 50ml cartridge, mixing tips type 6	REF 2504
pack, shade A3 50ml cartridge, mixing tips type 6	REF 2505
pack, shade A3.5 50ml cartridge, mixing tips type 6	REF 2506
pack, shade B1 50ml cartridge, mixing tips type 6	REF 2507
pack, shade B3 50ml cartridge, mixing tips type 6	REF 2508
pack, shade C2 50ml cartridge, mixing tips type 6	REF 2509
pack, shade BL 50ml cartridge, mixing tips type 6	REF 2510
bulk pack, shade A2 5 x 50ml cartridges, mixing tips type 6	REF 2512
bulk pack, shade A3 5 x 50ml cartridges, mixing tips type 6	REF 2513
mixing tips for Structur 3, 50 pieces, type 6	REF 2185
dispenser type 2	REF 2158
pack, shade A1 5ml QuickMix syringe, mixing tips type 10	REF 2515
pack, shade A2 5ml QuickMix syringe, mixing tips type 10	REF 2516
pack, shade A3 5ml QuickMix syringe, mixing tips type 10	REF 2517
pack, shade B1 5ml QuickMix syringe, mixing tips type 10	REF 2518
mixing tips for Structur 3, 50 pieces, type 10	REF 2189

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. **VOCO** recognizes its responsibility to replace products if proven to be defective. **VOCO** does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.
 For dental use only.**

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH