



VisCalor Dispenser

PREHEATING DISPENSER FOR COMPOSITE CAPS

VisCalor® Dispenser

Pre-warming
1 Cap in
only 30 Sec.



PREHEATING DISPENSER FOR COMPOSITE CAPS

With the VisCalor Dispenser, composite capsules can not only be warmed up, but they can also be applied immediately afterwards without the need to change devices, and their temperature can be maintained for a certain period of time. The composite is warmed up very quickly using near-infrared technology.

The VisCalor dispenser is the perfect device to use in combination with VisCalor bulk. This nanohybrid composite has been specially developed for warming up, and the dispenser application gives the material a lower viscosity, allowing it to flow optimally on margins and undercut regions. This also prevents air bubbles and minimises the risk of marginal gaps.

Advantages

- Effective handling – rapid warming and immediate application in one device
- Homogeneous warming thanks to near-infrared technology
- Two programs (VisCalor bulk and additional VOCO composites)
- Handy design and proven shape for filling molar cavities
- Better adaptation of composites to cavity walls thanks to silkier consistency

	Temperature	Pre-warming	Warm-keeping period
Setting 1 VisCalor bulk	65 °C	30 sec.	2:30 sec.
Setting 2 VisCalor bulk	65 °C	70 sec.	1:50 sec.

Presentation

- REF 6062 VisCalor bulk – VisCalor Dispenser Kit 80 x 0.25 g (16 x universal, 16 x A1, 32 x A2, 16 x A3.5), VisCalor Dispenser
- REF 6106 VisCalor – VisCalor Dispenser Kit 80 x 0.25 g (16 x A1, 32 x A2, 16 x A3, 16 x A3.5) VisCalor Dispenser
- REF 9143 VisCalor Dispenser – Preheating device (incl. charging cable, stand, 2 changeable tops and maintenance kit)
- REF 9144 Changeable tops 2 pcs. (autoclavable)
- REF 9145 Protective foils 100 pcs.



VOCO America Inc.
1245 Rosemont Drive
Suite 140
Indian Land · SC 29707
www.vocoamerica.com
infousa@voco.com
Toll-free phone: 1-888-658-2584
Fax: 1-888-849-3989